



Shipping Frozen Samples To VMRD Services

- 1) Package sample in Styrofoam
- 2) Add 3lbs of dry ice per day of transit time plus 3lbs extra.
 - a. Example: 4 days= 15lbs dry ice
- 3) Ship fastest way available (Mondays and Tuesdays work best). This will help to avoid unnecessary stops in transit over weekends.
- 4) Include Sample Submission Form, USDA Import Permit, and a Health Certificate to include information requested in the Import Permit
- 5) Use following HTS codes:
 - a. FBS: 3002.10.0240
 - b. All others: 3002.10.0290